# •• Blender Buddy

## Bio-Med Devices Blender

ASSEMBLY INSTRUCTIONS &  
INSTRUCTIONS FOR USE

R219P14-001, R219P14-002  
& R219P19

Maxtec, Inc.  
6526 South Cottonwood Street  
Salt Lake City, Utah 84107  
USA













TEL (800) 748.5355  
FAX (801) 270.5590  
www.maxtecinc.com

Authorized Representative:

QNET BV  
Hommerterweg 286  
6436 AM Amstenrade  
The Netherlands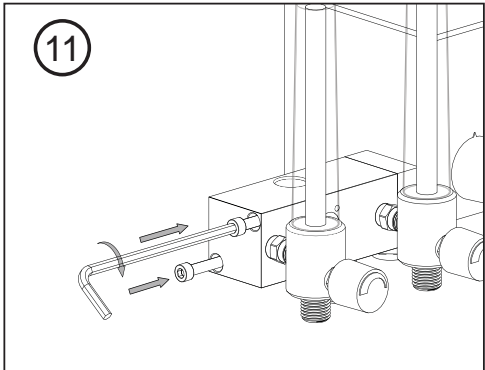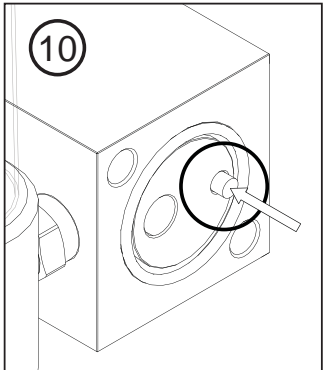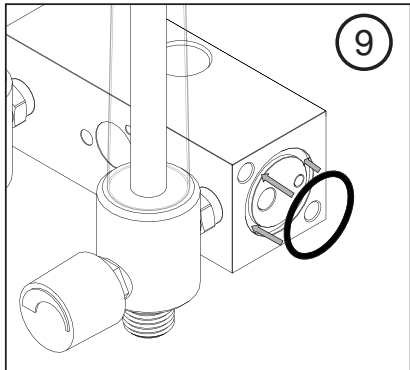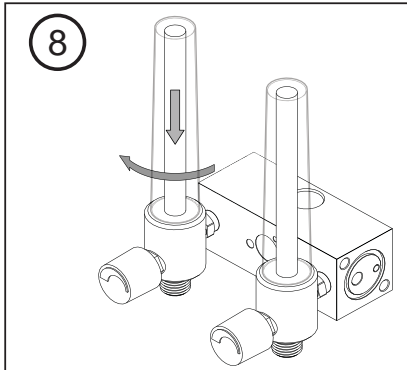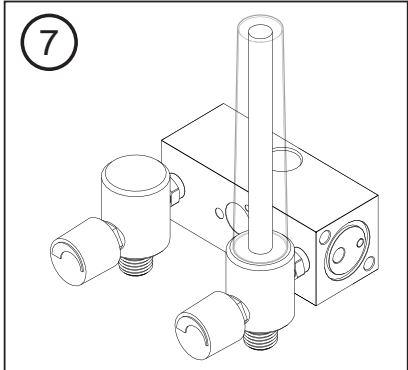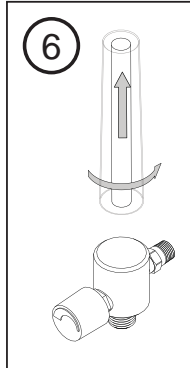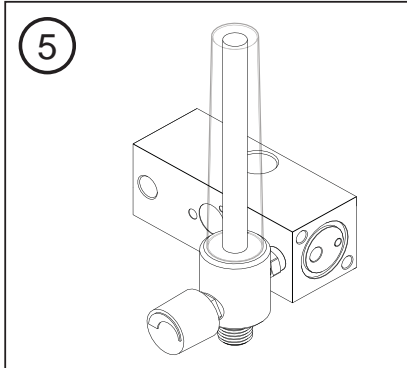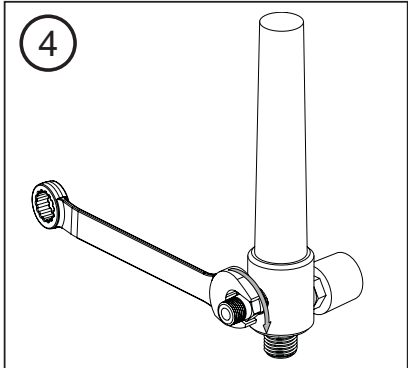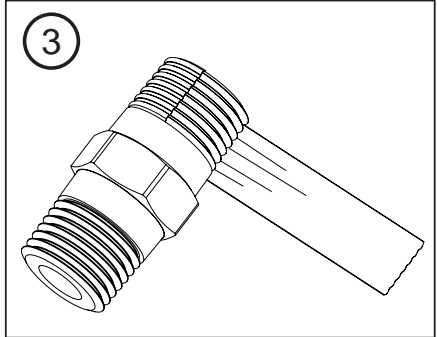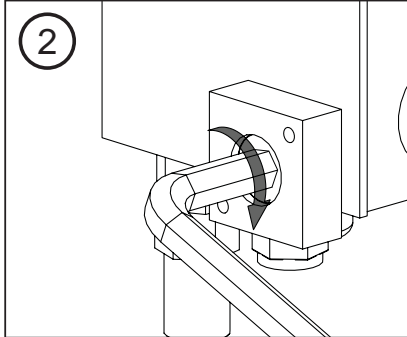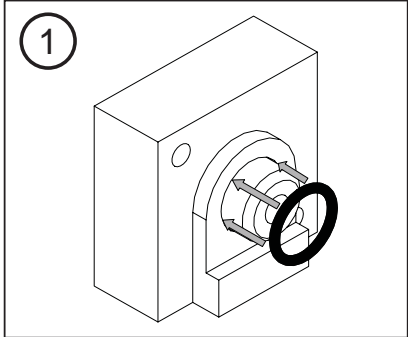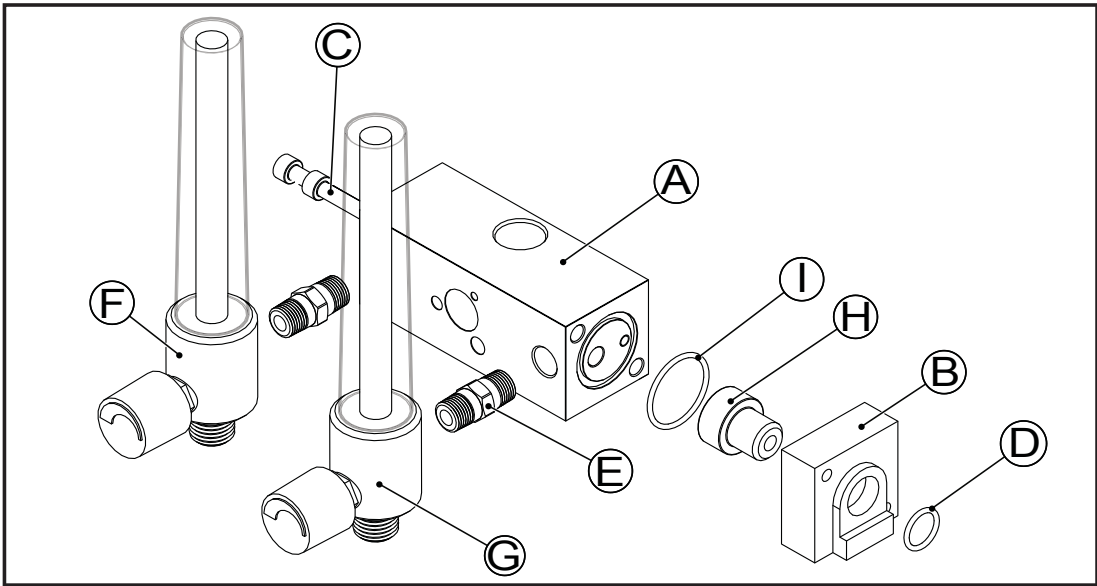

**For use with the following models:**  
 Biomed Devices Blenders  
 (2003, 2004)

## ❖ INSTRUCTIONS FOR USE

1. Be sure the small o-ring (D) is present around the threads of the short bolt (H) between the small block (B) and the blender.
  2. Attach the small adapter block (B) to the left side of the blender using the short bolt (H) and provided hex key. Remove any fittings from the left side blender outlet if necessary.
- \*Skip to Step 9 if flow meters are already attached.
3. Wrap both threaded ends of the pipe nipples (E) with teflon tape (1-1/2 wraps).
  4. Use a 7/16" wrench to tighten the pipe nipples (E) into the threaded holes of the flow meters (F) (G).
  5. Tighten one assembled flow meter (G) into one of the threaded holes on the large block (A).
  6. Remove the clear flow tube and shield from the other flow meter (F). Note how the flow meter components are arranged for reassembly in a subsequent step.
  7. Tighten the disassembled flow meter (F) into the other threaded hole in the large block (A).
  8. Reassemble the flow meter (F).
  9. Insert large o-ring (I) into o-ring groove of the large block (A).
  10. Ensure white material is inside the bleed port on the end of the large block.
  11. Attach large block (A) to small block (B) with the two long screws (C).
  12. Supply gas to the blender and check for leaks and proper function.

## ❖ WARNINGS

- » Carefully read instructions provided with flow meters, as well as this insert, before use.
- » For use with air/oxygen blenders only.
- » Max pressure is 100 psi.
- » Do not use lubricants on o-rings.
- » Check for leaks after assembly.
- » Not for use in MRI Environments.
- » Maxtec assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- » Replace EPDM o-rings during recommended blender rebuild. Use Maxtec approved o-rings.
- » This product should only be used under proper supervision of a healthcare professional.
- » If components are damaged or missing, contact your dealer immediately.
- » DO NOT use substitute parts.
- » Clean often using a cloth moistened with 65% isopropyl alcohol/water solution or germicidal wipe.
- » Follow all recommendations provided in the manual for the connected blender.

## ❖ WARRANTY

- » 2-year limited warranty on manufactured defects.

Manufacturer

Authorized Representative in the European Community













Lot code/Batch code

Catalog Number

Attention, consult accompanying documents

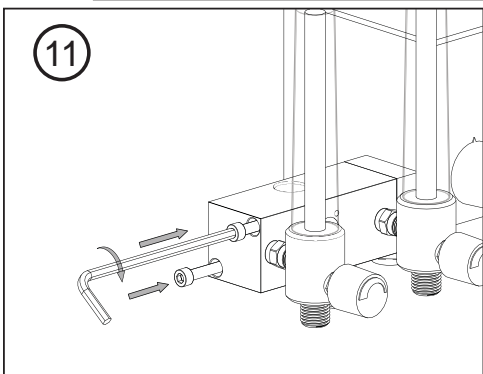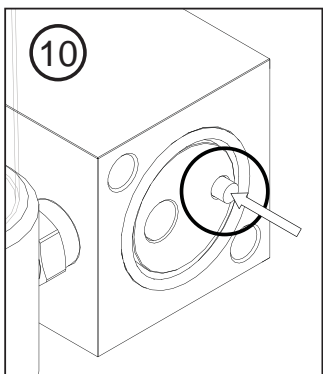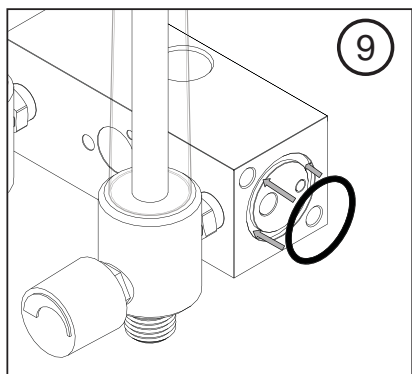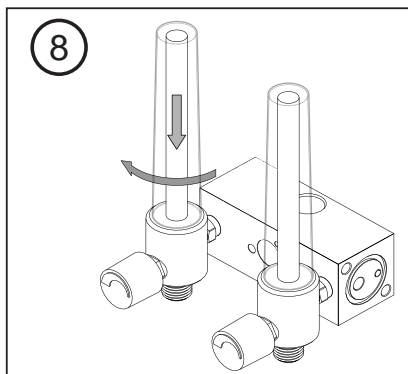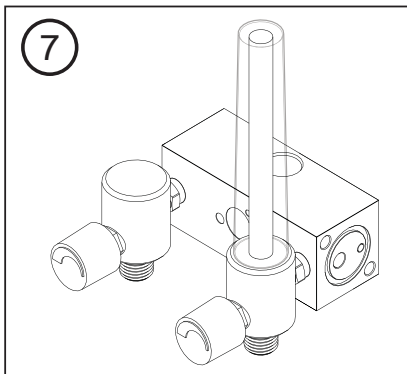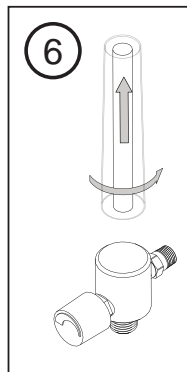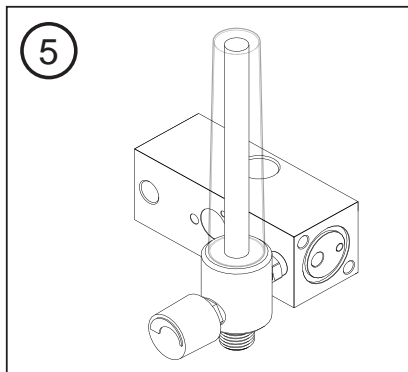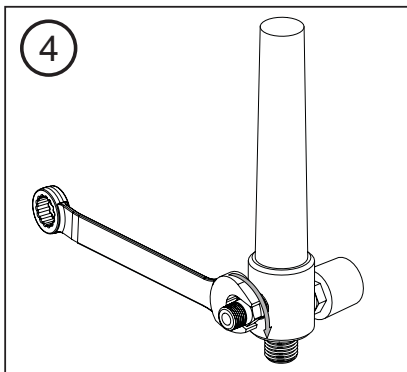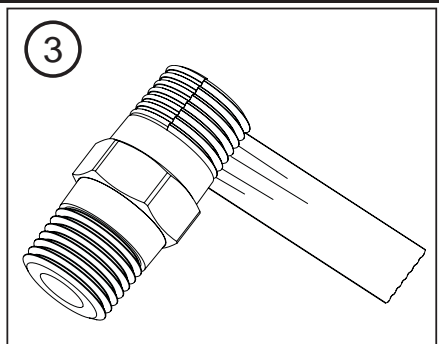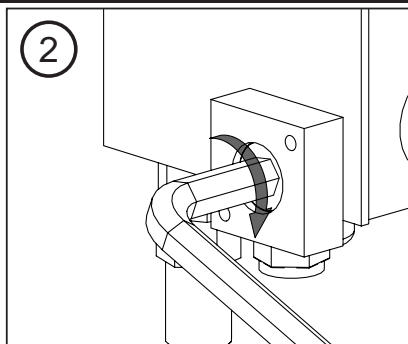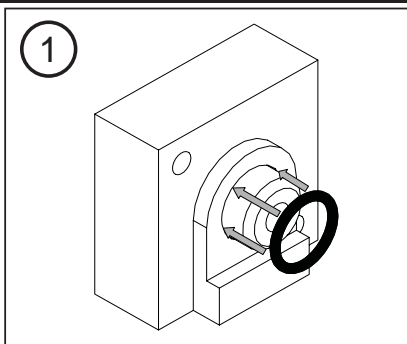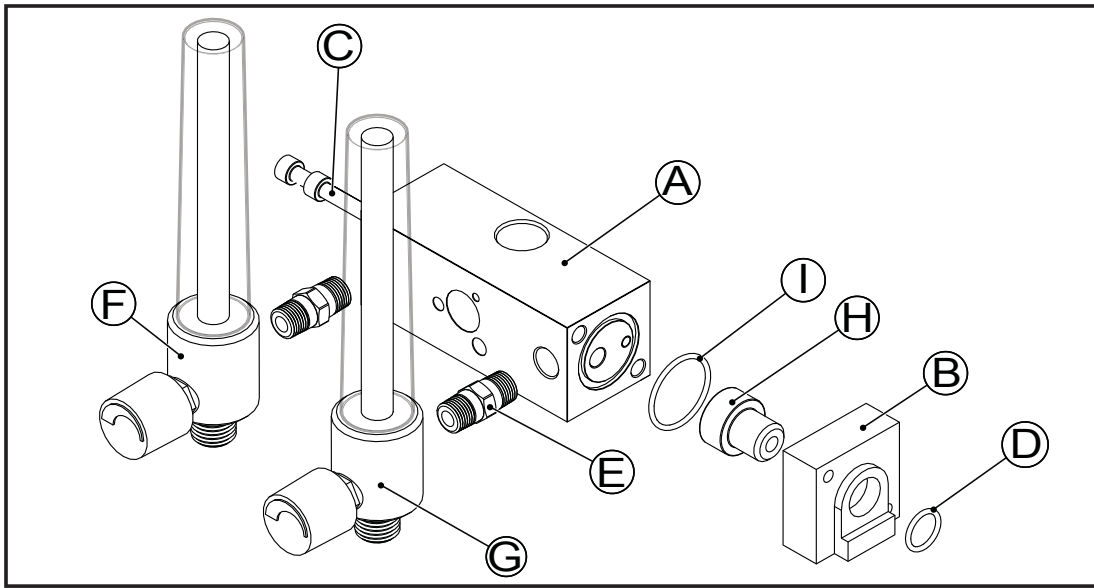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











Code de lot/code batch

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Attention, consultez les documents  
joints

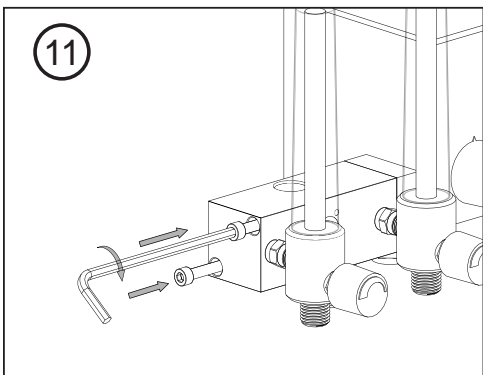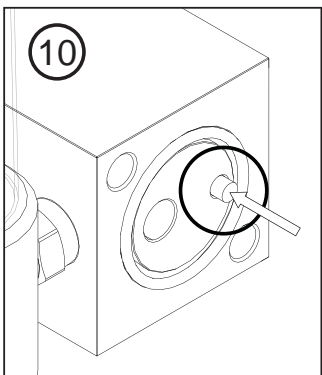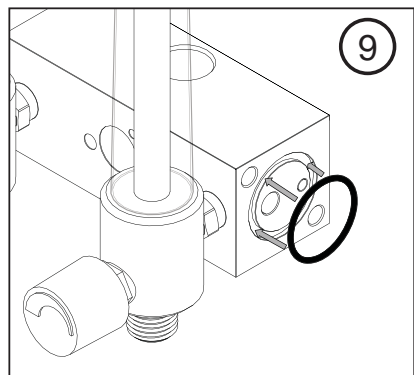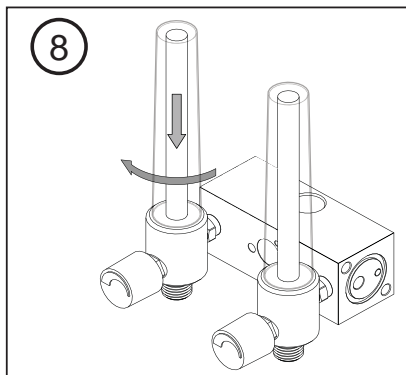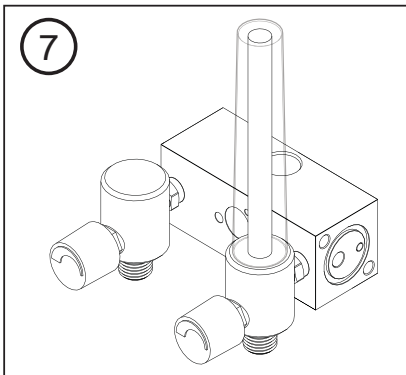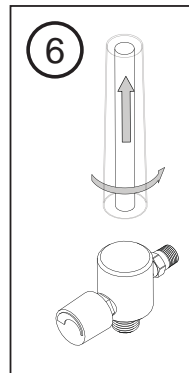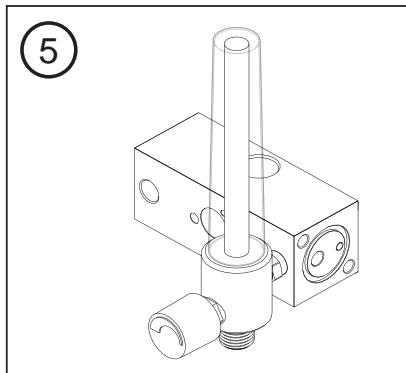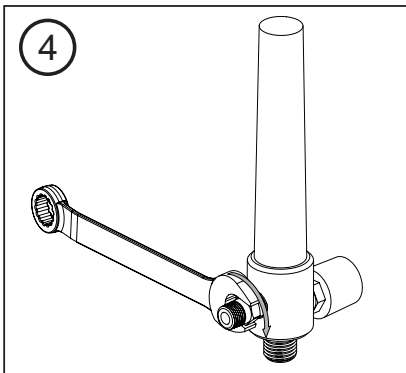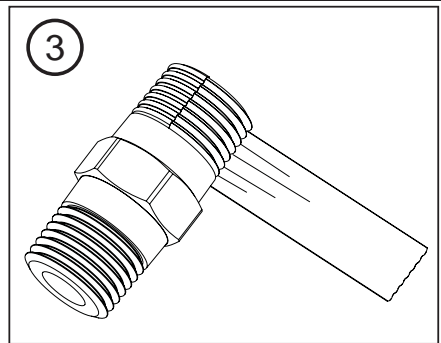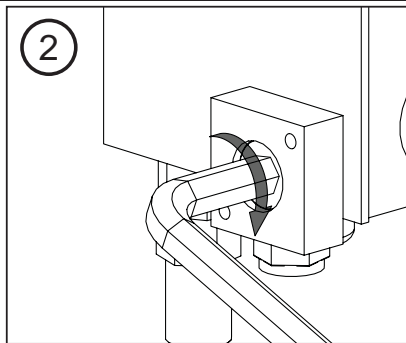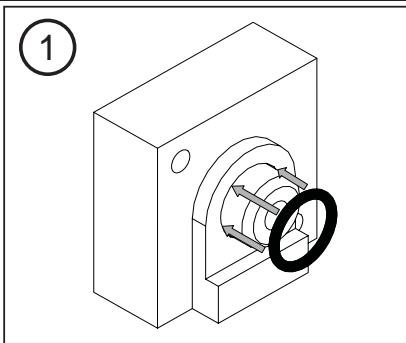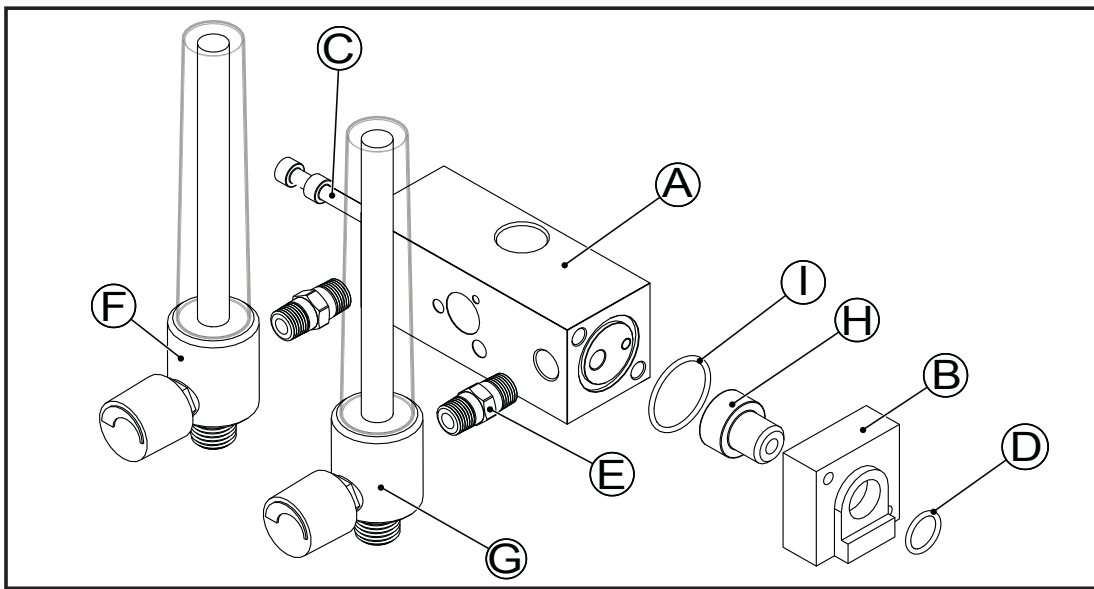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