# ∴ Blender Buddy

## Precision Medical Blender

ASSEMBLY INSTRUCTIONS &  
INSTRUCTIONS FOR USE

R219P20

Maxtec, Inc.  
6526 South Cottonwood Street  
Salt Lake City, Utah 84107  
USA

TEL (800) 748.5355  
FAX (801) 270.5590  
www.maxtecinc.com

Authorized Representative:

QNET BV  
Hommerterweg 286  
6436 AM Amstenrade  
The Netherlands

## ❖ INSTRUCTIONS FOR USE

1. Attach the DISS female fitting (A) of the Blender Buddy to the DISS male fitting (B) on the left side of the Precision Medical Blender (E).
2. Attach flow meters (C) to the male DISS fittings (D) on the front of the Blender Buddy.
3. Supply gas to the blender and check for leaks and proper function.

## ❖ WARNINGS

- » Carefully read instructions provided with flow meters, as well as this insert, before use.
- » For use with air/oxygen blenders only.
- » Max pressure is 100 psi.
- » Do not use lubricants on o-rings.
- » Check for leaks after assembly.
- » Not for use in MRI Environments.
- » Maxtec assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- » Replace EPDM o-rings during recommended blender rebuild. Use Maxtec approved o-rings.
- » This product should only be used under proper supervision of a healthcare professional.
- » If components are damaged or missing, contact your dealer immediately.
- » DO NOT use substitute parts.
- » Clean often using a cloth moistened with 65% isopropyl alcohol/water solution or germicidal wipe.
- » Follow all recommendations provided in the manual for the connected blender.

## ❖ WARRANTY

- » 2-year limited warranty on manufactured defects.

Manufacturer

Authorized Representative in the European Community

Lot code/Batch code


Catalog Number

Attention, consult accompanying documents

**For use with the following models:**  
Precision Low Flow Blender  
(PM5300)

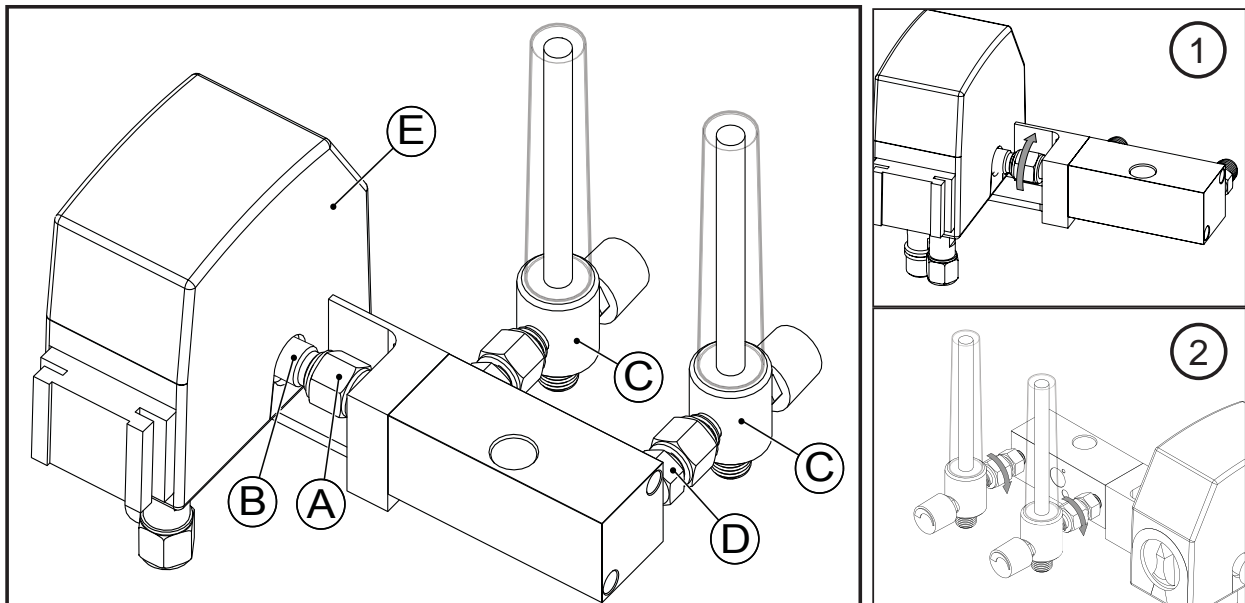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Code de lot/code batch

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Attention, consultez les documents joints

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