6526 South Cottonwood Street
 Salt Lake City, UT 84107
www.maxtecinc.com
 866.4.maxtec

- ⚠ Warning: This device is not MRI compatible.
- ⚠ Warning: Never exceed 100 lbs on this device.
- ⚠ Warning: Do not attach more than 15 lbs on one single hook. Distribute weight evenly.
- ⚠ Warning: Do not mount equipment that places the load more than 5 inches off center of the pole.
- ⚠ Warning: Do not mount more than 25 lbs on upper portion of this device.
- ⚠ Warning: Use caution when rolling heavily loaded device, especially on uneven surfaces.
- ⚠ Warning: This product should only be used under proper supervision from a healthcare professional.
- ⚠ Warning: Use caution when adjusting pole height with heavy objects mounted above hands.
- ⚠ Warning: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts unless provided by Maxtec or authorized representative.

- ⚠ Warning: Do not autoclave.
- ⚠ Warning: Do not stand on or ride this device.
- ⚠ Warning: Brakes may not stop IV pole on all surfaces.
- ⚠ Warning: Follow the order of assembly as directed.
- ⚠ Warning: Failure to adhere to these warnings can result in severe injury.
- ⚠ Warning: Maxtec assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- Cleaning:
 Clean often using a cloth moistened with 65% isopropyl alcohol/water solution or germicidal wipe.
 Clean the entire device before use.
- Warranty: 1-year limited warranty on manufactured defects.

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