

❖ Exhalometer & F.V.C.



- **How to measure F.V.C (Forced Vital Capacity) with the Exhalometer**

Measuring Forced Vital Capacity:

1. Power the Exhalometer off, then back on.
2. 0 RESP/MIN and 0.0 EXP.MIN.VOL displays will alternate on the LCD display
3. Ask patient to take a maximum breath and then exhale fully through the Exhalometer.
4. Remove the Exhalometer from the circuit to prevent the patient from exhaling through it again.
5. Forced Vital Capacity (F.V.C.) will be displayed on the LCD. (Display will alternate between 1 RESP/MIN and the FVC. The value of which will be displayed as the EXP.MIN.VOL.)

Part Numbers

Exhalometer	R501P01	Filter 19mm	R501P03
Filter 30mm	R501P02	Adapter	R501P06
Carrying Case	R213P02	Exhalometer Sanitizing Spray	R501P07