SmartStack

Premium

ASSEMBLY INSTRUCTIONS &
INSTRUCTIONS FOR USE

R100P49

Maxtec
6526 South Cottonwood Street
Salt Lake City, Utah 84107
USA

TEL (800) 748.5355
FAX (801) 270.5590
email: sales@maxtecinc.com
website: www.maxtecinc.com





Patent Pending

Authorized Representative:

QNET BV
Hommerterweg 286
6436 AM Amstenrade
The Netherlands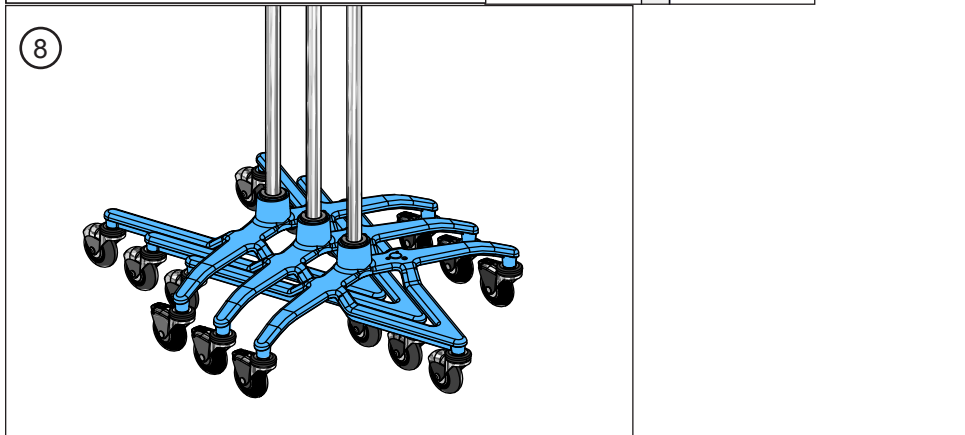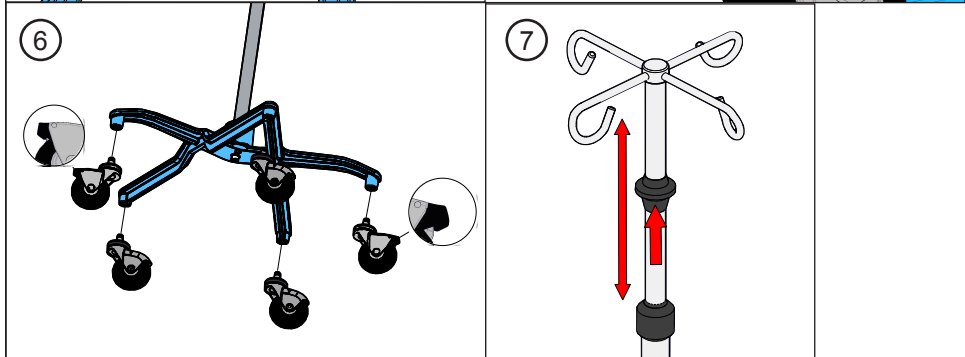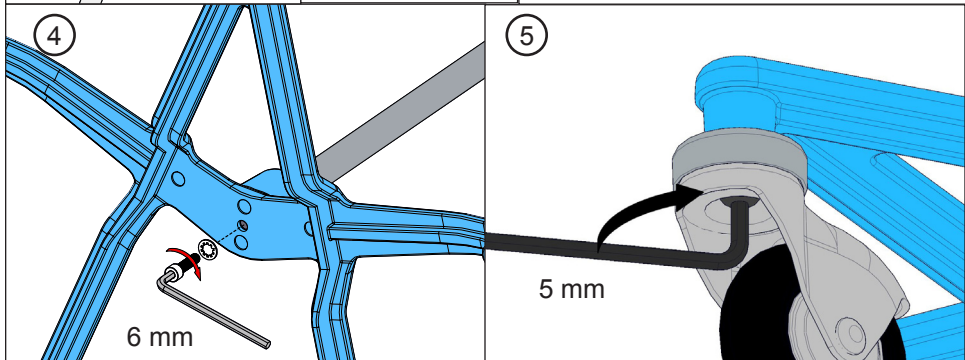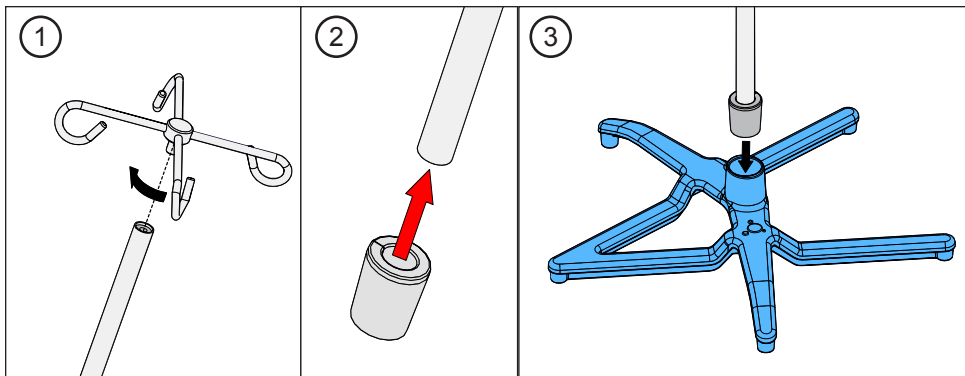

⚠️ WARNINGS

- » **This device is not MRI compatible.**
- » Never exceed 100 lbs on this device.
- » Do not attach more than 11 lbs (5kg) on one single hook. Distribute weight evenly.
- » Do not mount equipment that places the load more than 5 inches off center of the pole.
- » Do not mount more than 50 lbs on upper portion of this device.
- » Use caution when rolling heavily loaded device, especially on uneven surfaces.
- » This product should only be used under proper supervision from a healthcare professional.
- » Do not autoclave.
- » Do not stand on or ride this device.
- » Brakes may not stop IV pole on all surfaces.
- » Follow the order of assembly as directed.
- » Use caution when adjusting pole height with heavy objects mounted above hands.
- » If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts unless provided by Maxtec or authorized representative
- » Failure to adhere to these warnings can result in severe injury.
- » Maxtec assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- » Use caution if IV pole is fully loaded. Product may tip.
- » Use caution around stairwells.

⚙️ CLEANING

- » Clean often using a cloth moistened with 65% isopropyl alcohol/water solution or germicidal wipe.
- » Clean the entire device before use.

⚙️ WARRANTY

- » 4-year limited warranty on manufactured defects.

Attention, consult accompanying documents

LOT

Lot code/Batch code

EC REP

Authorized Representative in the European Community

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Attention, consultez les documents joints

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